

Refurbished with 90 day warranty \$795.00 with trade-in

PD-700

Professional Medical Sales
520 N Parramore Avenue
Orlando, FL 32801

407-843-9797



613-R DYNACLAVE

STEAM STERILIZATION FOR:

- MEDICAL OFFICE
- SMALL LABORATORY
- INDUSTRIAL
- FIRST AID ROOM
- DENTAL LAB
- VETERINARIAN



Office Sterilizers



TWO TRAYS
ARE FURNISHED
THIRD IS OPTIONAL

EVERY OFFICE CAN HAVE THE BENEFITS
OF PRESSURE STEAM STERILIZATION
WITH THE LOW-COST

AMSCO 613-R DYNACLAVE

The 613-R Dynaclave makes a significant contribution to the solution of the physician's problem of satisfying two opposing pressures — the increasingly complex demands of patient care and protection . . . and an efficient, yet economically sound, office operation. A singular justification for the small investment in a 613-R Dynaclave is the **important contribution it can make toward patient protection.** Other benefits add up to time-savers for busy, and costly, office personnel. Sterilizing temperature may be either 250 or 270 F . . . the 613-R heats to 270 F in ten minutes. Its capacity means fewer loads, sterile supplies (processed during slack hours) are there when needed, and it's easy to operate. Instruments and supplies are cooled and dried by exhausting steam and residual water back into water reservoir — NOT into room.

DIMENSIONS Overall — 15¼" high, 12¼" wide, 17¾" deep (from front of door handle).

Capacity — Usable chamber 5¾" x 6¼" x 13". Equipped with two noncorrosive metal trays 13" long, 5½" wide, 7/8" deep (additional trays optional). Greater capacity means more output with fewer loads in less time.

CONSTRUCTION **Door** — Noncorrosive safety lock door and bar with built-in gasket — sturdy, safe, and easy operation.

Jacket — Noncorrosive stainless steel with covered opening to water reservoir . . . easy to clean.

Heater — 1400 watt hairpin type, thermostat-controlled with manual reset low-water cutoff device for burnout protection.

Reservoir — Holds ample water for sterilization plus capacity to accept steam condensate and residual water when exhausting the chamber.

FEATURES **U. L. listed** — 120 volt, 60 Hz or 240 volt, 60 Hz alternating current. Just plug in the 6 ft. cord and it's ready to operate.

On-off Switch — for positive control. Easy-to-understand operating instructions are attached to top of unit.

Signals — Red light indicates heater is on — white light indicates heater is off.

Valve — Handle in "down" position fills chamber with proper amount of water. In "up" position, exhausts steam and residual water. At "level" position water supply and steam exhaust lines are locked.

Air Elimination — Air and entrained vapor are automatically and continuously evacuated from the chamber thru the air eliminator trap and condenser coil.

Timer — Automatic 60-minute timer controls a selected cycle from 3 to 60 minutes.

Thermometer — Dial face shows chamber temperature within ± 2 F. Provides visual proof of proper temperature for selected cycle and type of load.

Weight — 35 pounds — as easy to move as your office typewriter.

This catalog is intended for the exclusive use of AMSCO customers, including architects or designers. Reproduction in whole or in part by others is prohibited.

© AMSCO — 1979-1980

AMSCO IS "AT YOUR SERVICE"

Authorized AMSCO Surgical Supply Dealers are strategically located throughout the country. The dealer near you is eager to be of service in the selection and maintenance of the right AMSCO equipment to meet the requirements of your office. For his name and address write to: